

# FC2 | FEMALE CONDOM

An alternative for women

UPC 61783-000-12  
NDC 61783-0000-12  
GTIN 00861783000126  
Case 1086178300123



- ✓ Consists of a nitrile (**non-latex**) sheath and outer ring, and a polyurethane inner ring
- ⊗ **No spermicidal additives**
- 💧 Lubricated with a **silicone-based lubricant** inside and out
- ✓ The **only FDA approved** for market female condom. Classified as a Class III medical device, proven safe and effective.

Source: [www.fda.gov](http://www.fda.gov),  
PMA number P080002

## What's NEW about FC2™?

FC2 is now available in a 12 Pack!

*Ask your Physician or Pharmacist to learn more*

**FC2 is a female condom designed to be inserted into the vagina.**

### COVERAGE

- FC2 Female Condom is an OTC form of contraception that is covered by most insurance companies with **\$0 out-of-pocket** costs when obtained with a prescription

RX Sample	
Patient Name:	_____
Rx:	FC2 Female Condom (1 box of 12 condoms)
Sig:	Use per packaging instructions with each act of intercourse
Disp:	_____ Refills: _____
_____, MD	

- **Pharmacists** can find this product in their database by searching either "FC2 Female Condom" or

**UPC 61783-000-12**  
**NDC 61783-0000-12**  
**GTIN 00861783000126**

**To order, contact your distributor**

### NO COMPROMISES

- **Non-hormonal and no side effects** or contraindications
- No medical procedures necessary
- **Latex-free**
  - Alternative for those with latex allergies or sensitivities
- **"Backup" Protection**
  - During the initiation of hormonal contraceptives
  - When changing contraceptive methods
  - For post-partum women
- **Female condoms are a source of pleasure for users** (*Dias et al. 2006*), and are less constrictive for men versus male condoms
- Can be inserted in advance, so there is no interruption during sex

### PROTECTION

- **Comparable to male condoms**, FC2 offers **dual protection** against STIs (including HIV) and unplanned pregnancy (*Bounds et al. 1992, Farr et al. 1994; Trussell 1998; Soper et al. 1993, Fontanet et al. 1998 Feldblum et al. 2002, French et al. 2003*)
- **FC2 offers highly effective protection** against STI transmission (including HIV), by providing a barrier which prohibits the exchange of (sexual) body fluids and reduces skin-to-skin contact



The Female Health Company  
**[www.fc2.us.com](http://www.fc2.us.com)**  
**[info@fc2.us.com](mailto:info@fc2.us.com)**

**f FC2 Female Condom USA**

**t FC2USA**

## STEP-BY-STEP INSTRUCTIONS


Once you get the hang of it, the FC2 Female Condom is easy to use. With a little practice you'll soon become very comfortable with it! Try it at least three times and you'll see why FC2 is a great choice for both women and men.



### Open the Packet


Locate arrow and carefully tear down from the notch on the top of the package.

**DO NOT USE SCISSORS OR KNIFE TO OPEN.**

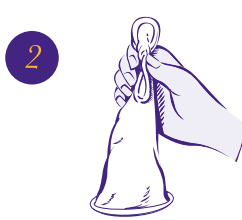


### How to Hold

Hold the condom at the closed end, open end hanging down. Grasp the flexible inner ring and squeeze with the thumb and middle finger so it becomes long and narrow.

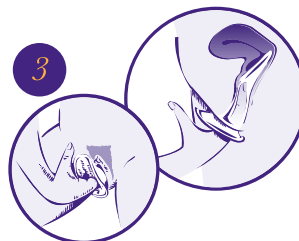


### Choose a Position and Insert

Squat, raise one leg, sit or lie down. Gently insert the inner ring into the vagina. Push the inner ring up and back into the vaginal canal, moving it into place. **Push the condom as far as it will go – near pubic bone.** Make sure it is not twisted. The outer ring remains outside the vagina.

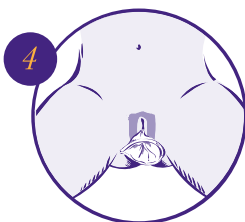

### Ready for Use

FC2 Female Condom is now in place and ready to use with your partner.

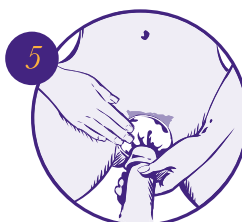

### Guide

Gently guide your partner's penis into the vagina. Make sure that it enters properly. Once the penis is inserted, if there is discomfort, withdraw the penis and reposition the inner ring (discomfort can be caused by the inner ring bumping against the cervix when FC2 is not properly inserted).



### Removal

To remove the condom twist the outer ring and gently pull the condom. Wrap the condom in the sachet package or tissue and throw into the garbage.

**DO NOT FLUSH. DO NOT REUSE.**

## PRODUCT DETAILS AND USE


- 1 The polyurethane inner ring is used for insertion and to hold the sheath in place during intercourse.



Made of Nitrile - Latex Free

The nitrile outer ring covers the area around the opening of the vagina.

- 2 Be sure that the penis is not entering on the side or between the sheath of the vagina wall. Make sure that the outer ring is not pushed into the vaginal opening, it should remain outside, covering the labia.

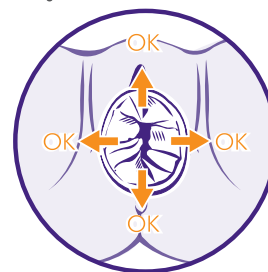

NOT OKAY

**IMPORTANT INFORMATION** \* Natural rubber latex condoms for men are highly effective at preventing sexually transmitted infections, including HIV/AIDS, if used correctly. \*If you are not going to use a male natural rubber latex condom, you can use the FC2 Female Condom to help protect yourself and your partner. \*The FC2 Female Condom only works when you use it. Use it correctly every time you have sex. \*Before you try the FC2 Female Condom, be sure to read the directions and learn how to use it correctly.

**RISKS OF USING THE FC2:** Use the FC2 Female Condom correctly every time you have sex. If the FC2 is not used correctly every time, your risk of getting a sexually transmitted infection or becoming pregnant will be higher. Incorrect use may result in the FC2 not providing protection. To avoid risk of incorrect use, read the instructions carefully prior to use. Research has shown that the following events are rare but can occur when using the FC2: discomfort during insertion, pain after insertion, before sex, discomfort during sex, burning sensation, rash or itching.

**BENEFITS OF USING THE FC2:** Using the FC2 during sexual intercourse can help protect against getting a sexually transmitted infection or getting pregnant. The FC2 does not contain natural rubber latex. The FC2 provides women with an option for protection which they can initiate. Women can insert the FC2 ahead of the sex act. This allows for a more spontaneous act because an erection is not needed to insert the FC2.

For more information please contact us at: [info@fc2.us.com](mailto:info@fc2.us.com) or 1 (800)-274-6601

[www.fc2.us.com](http://www.fc2.us.com)