



# FC2™ EMPOWERS WOMEN

## TAKE CONTROL OF YOUR SEXUAL PROTECTION AND PLEASURE

- ✔ **THE ONLY FDA APPROVED** for market female condom
- ↻ **DUAL PROTECTION** in preventing both pregnancy and STIs
- ⊘ **HORMONE FREE** for women looking for drug free options
- 💰 **FC2 IS COVERED** with \$0 co-pay by most insurance plans when prescribed by a physician
- ⊘ **LATEX FREE** for those with allergies or sensitivities
- ♥ **SOURCE OF PLEASURE** for both users
- 💧 **PRE-LUBRICATED** inside and out

Try FC2 because we've  
got you covered!



✉ [INFO@FC2.US.COM](mailto:INFO@FC2.US.COM)

🌐 [WWW.FC2.US.COM](http://WWW.FC2.US.COM)

📘 [FC2 THE FEMALE CONDOM](#)

🐦 [FC2USA](#)

# STEP-BY-STEP INSTRUCTIONS

THE FC2 FEMALE CONDOM® IS EASY TO USE.  
WITH A LITTLE PRACTICE YOU'LL SOON BECOME VERY COMFORTABLE WITH IT!



## OPEN THE PACKET

Carefully tear down from the notch.  
**DO NOT USE SCISSORS OR KNIFE TO OPEN.**




## HOW TO HOLD

Hold at the closed end, open end hanging down. Grasp the flexible inner ring and squeeze.




## CHOOSE A POSITION & INSERT

Squat, raise one leg, sit or lie down. Gently insert inner ring into vagina. Push up and back. Be sure it is not twisted. The outer ring remains outside.

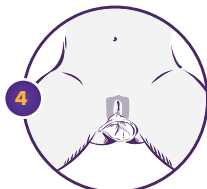

## READY FOR USE

FC2 Female Condom is now in place and ready to use with your partner.

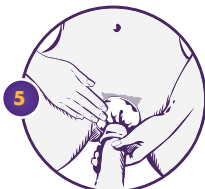

## GUIDE

Gently guide your partner's penis into the vagina.



## REMOVAL

Twist the outer ring and gently pull. **DO NOT FLUSH. DO NOT REUSE.**

## FOR MORE INFORMATION:

✉ [INFO@FC2.US.COM](mailto:INFO@FC2.US.COM)

🌐 [WWW.FC2.US.COM](http://WWW.FC2.US.COM)

📘 FC2 THE FEMALE CONDOM

🐦 FC2USA