



FC2[®] FREEDOM TO CHOOSE YOUR PROTECTION

TAKE CONTROL OF YOUR SEXUAL PROTECTION AND PLEASURE

- ✓ **THE ONLY FDA APPROVED** for market internal condom
- ↻ **DUAL PROTECTION** in preventing both pregnancy and STIs
- ⊘ **HORMONE FREE** for women looking for drug free options
- 💰 **FC2 IS COVERED** for most insured women when prescribed
- ⊘ **LATEX FREE** for those with allergies or sensitivities
- ♥ **A POSSIBLE SOURCE OF PLEASURE** for both users
- 💧 **PRE-LUBRICATED** inside and out

Try FC2 because we've
got you covered!



✉ INFO@FC2.US.COM
🌐 WWW.FC2.US.COM
LET GET SOCIAL: @FC2USA

V11001

STEP-BY-STEP INSTRUCTIONS

THE FC2[®] INTERNAL CONDOM IS EASY TO USE.
WITH A LITTLE PRACTICE YOU'LL SOON BECOME VERY
COMFORTABLE WITH IT!



OPEN THE PACKET

Carefully tear down from the notch.
DO NOT USE SCISSORS OR KNIFE TO OPEN.




HOW TO HOLD

Hold at the closed end, open end hanging down. Grasp the flexible inner ring and squeeze.




CHOOSE A POSITION & INSERT

Squat, raise one leg, sit, or lie down. Gently insert the inner ring into vagina. Push up and back. Be sure it is not twisted. The outer ring remains outside.

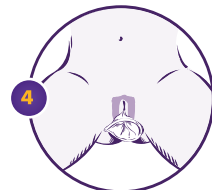

READY FOR USE

FC2 Internal Condom is now in place and ready to use with your partner.

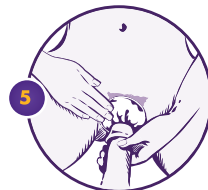

GUIDE

Gently guide your partner's penis into the vagina.



REMOVAL

Twist the outer ring and gently pull. DO NOT FLUSH. DO NOT REUSE.

FOR MORE INFORMATION:

✉ INFO@FC2.US.COM 🌐 WWW.FC2.US.COM LET'S GET SOCIAL: @FC2USA

Before using FC2, please read the product leaflet and complete product labeling information: www.fc2.us.com/labeling.



The Female Health Company is a business unit of Veru Inc. Miami, FL 33137.
The FC2 name and logo are registered trademarks of Veru Inc. All rights reserved ©2019.

V11001