

# FC2 INTERNAL CONDOM

## WHAT is the FC2 Internal Condom?

It is the second-generation internal condom which is a soft, smooth, strong and safe condom made from nitrile, which is worn inside the vagina. It is a transparent sheath that is about 17 centimeters or about 6.5 inches long which lines the walls of the vagina, allowing the penis to move freely inside the condom during sex.



## HOW is FC2 worn?

FC2 has an inner and an outer ring. The inner ring is used to insert FC2 and to keep the condom in place during sex. The outer ring remains outside the vagina. Just like a male condom, the FC2 is a barrier to sperm after ejaculation. The difference is that the FC2 internal condom is a female initiated latex free available method of protection from unplanned pregnancy and STI's, including HIV.

## WHY use FC2?

FC2 provides protection from pregnancy and STIs in one package. Insertion can be in advance of sex and removal can be delayed until after sex.

- **For anybody**
  - FC2 fits all women and men regardless of their size and shape.
  - FC2 is non-allergenic; it may be suitable for men and women who have a sensitivity to latex.
- **In control**
  - You don't need to see a doctor before you start using internal condoms.
  - FC2 is a non-hormonal contraceptive option.
  - FC2 may provide extra protection by covering the woman's external sex organs and the base of the penis.
- **Pleasurable to use**
  - FC2 is made from nitrile, which means that - as opposed to latex condoms - it can be used in combination with oil-, water-, or silicone-based lubricants.
  - FC2 heats up to body temperature which could make sex with FC2 feel more warm and natural.
  - FC2 is lubricated with a high-quality silicone-based lubricant.
  - For some users, the two rings can provide extra stimulation.

## WHEN to insert FC2?

- FC2 can be inserted either a few hours or just before sex.
- The penis doesn't need to be erect to use FC2 and doesn't have to be withdrawn immediately after sex.

## WHO can use FC2?

- FC2 is indicated for vaginal use for the prevention of unplanned pregnancy and sexually transmitted infections (STIs). It can be used by anyone who wishes to have safe sex.
- In addition, FC2 is suitable for women who are menstruating, pregnant, or (pre-or post) menopausal.
- FC2 can be used with the pill, injections, intrauterine device (IUD), implants, post sterilization and post vasectomy to provide protection against STIs.

Before using FC2, please read the product leaflet and complete product labeling information: [www.fc2.us.com/labeling](http://www.fc2.us.com/labeling)

## Want to learn more?

[www.fc2.us.com](http://www.fc2.us.com) | [info@fc2.us.com](mailto:info@fc2.us.com) | [f](#) [@](#) [FC2USA](#) | [FC2 INTERNAL CONDOM](#)

The Female Health Company is a business unit of Veru Inc. Miami, FL 33137.  
The FC2 name and logo are registered trademarks of Veru Inc. All rights reserved © 2019.

## HOW TO USE FC2?



Spread the lubricant



Open the package



Hold and squeeze the inner ring



Insert the internal condom



You are ready to have safe sex

